



www.ergotron.com

USA 1-800-888-8458

Europe +31 (0)33-45 45 600

China 86-769-86018920


Change Power System Batteries

40 Ah Electrical System • SV32-32001-S
• SV32-32002-S

66 Ah Electrical System • SV32-32025-S
• SV32-32026-S
• SV32-32051-S
• SV32-32052-S

• SV32-32025-T
• SV32-32026-T
• SV32-32051-T
• SV32-32052-T

- SV32-32003
- SV32-32004
- SV32-32005
- SV32-32006
- SV32-32007
- SV32-32008
- SV32-32009
- SV32-32010
- SV32-32011
- SV32-32012
- SV32-32053
- SV32-32054
- SV32-32055
- SV32-32056
- SV32-32057
- SV32-32058
- SV32-32059
- SV32-32060
- SV32-32061
- SV32-32062

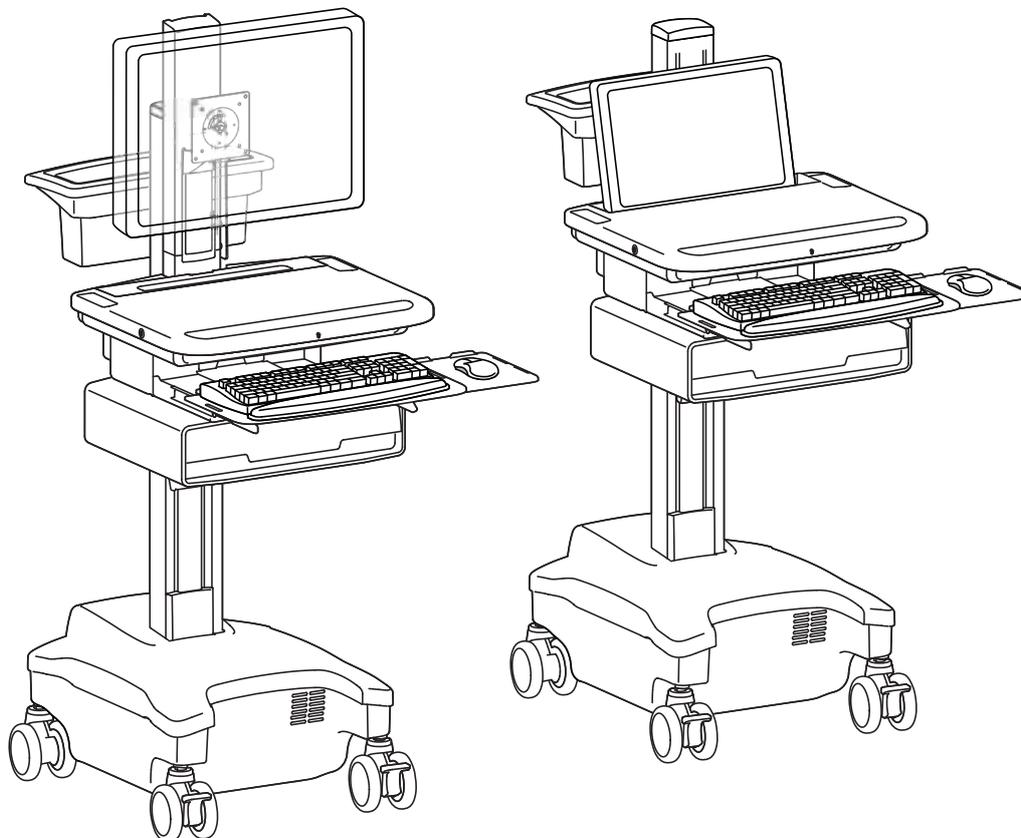


For the latest User Installation Guide and StyleLink Software Download please visit:

Encontrará la versión más reciente del manual de instalación del usuario y el software de StyleLink en:
 Si vous souhaitez télécharger le dernier manuel d'installation de l'utilisateur ou le logiciel StyleLink, rendez-vous sur:
 Den neuesten Installationsleitfaden für Benutzer sowie den neuesten StyleLink-Software-download finden Sie unter:
 Voor de nieuwste Installatiehandleiding voor de gebruiker en voor het downloaden van StyleLink-software gaat u naar:
 Per scaricare le versioni più recenti del manuale di installazione e del software StyleLink, andare al sito:
 最新版ユーザーインストールガイドとスタイルリンクソフトウェアは次のサイトでダウンロードできます。
 请从下列地址获取最新版本的用戶安裝指南和 StyleLink 軟件下載：
 StyleLink

<http://4support.ergotron.com>



- User's Guide - English
- Guía del usuario - Español
- Manuel de l'utilisateur - Français
- Gebruikersgids - Nederlands
- Benutzerhandbuch - Deutsch
- Guida per l'utente - Italiano
- ユーザーガイド : 日本語
- 用戶指南 : 汉语

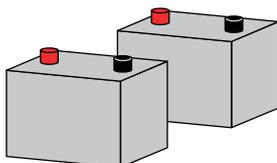


Two 20Ah or two 33Ah Sealed Lead Acid, Absorbed Glass Mat, 12VDC batteries.




LEAD BATTERY

Please dispose of all batteries in accordance with local law

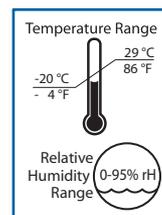

Operational

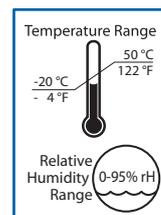

Storage



10mm (3/8")
or
8mm (5/16")

Depends on battery



WARNING:

ALWAYS REPLACE BOTH BATTERIES. REPLACE BATTERY WITH SAME AMP/HOUR RATED BATTERY ONLY.
 Only Ergotron-specified batteries may be used in the StyleView Power System. Please call customer care for more details.
 Do not remove or replace the batteries while cart is located in an oxygen rich or hazardous environment. Sparking may occur.
 Recycle battery or contact Ergotron for proper battery disposal guidelines.



20 Amp-hour Batteries

WARNING

IMPORTANT REPLACEMENT BATTERY INFORMATION

The power module is configured for two, 20 A-h batteries. Only the following batteries are compatible with this system:

- B&B EVP20-12 (I2)
- SBS S-12220 (F15)
- Longway 6FM20G

Installing batteries other than the 20 A-h listed above will void the product warranty. Failure to heed this warning may result in severe damage to batteries, power module and possible fire hazard. Contact Ergotron for more information at 1-800-888-8458.

820-956

33 Amp-hour Batteries

WARNING

IMPORTANT REPLACEMENT BATTERY INFORMATION


The power module is configured for two, 33 A-h batteries. Only the following batteries are compatible with this system:

- B&B EP33-12 (I2)
- SBS S-12330 (F15)
- Longway 6FM33G
- Vision 6FM33D


Installing batteries other than the 33 A-h listed above will void the product warranty. Failure to heed this warning may result in severe damage to batteries, power module and possible fire hazard. Contact Ergotron for more information at 1-800-888-8458.

820-957

1 Turn off all mounted equipment.

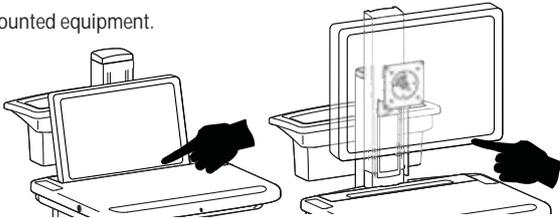


2 Disconnect Power System from power source.

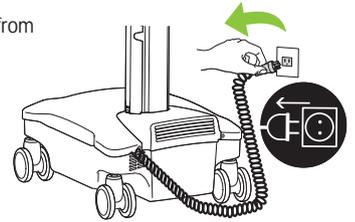


3


Turn power system off by holding down the AC Outlet Power button for 1 - 3 seconds. Power light will shut off.

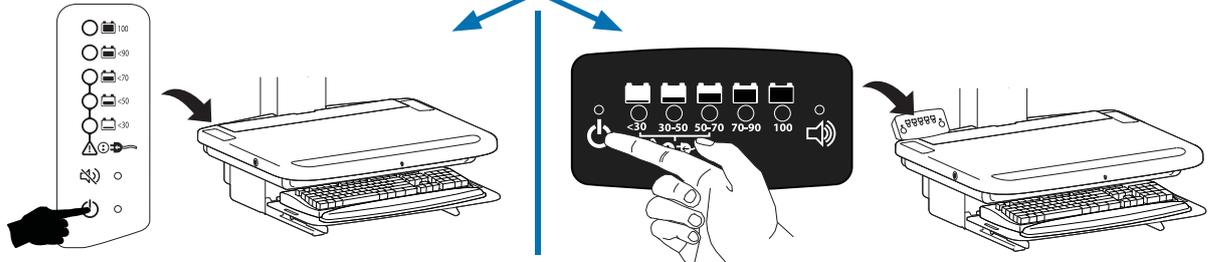


4 a

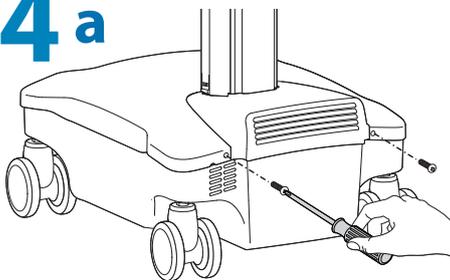


b

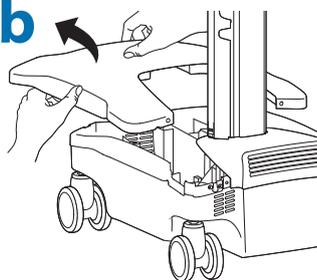


c

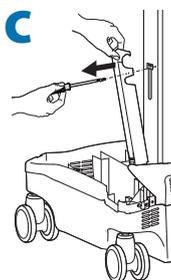


d

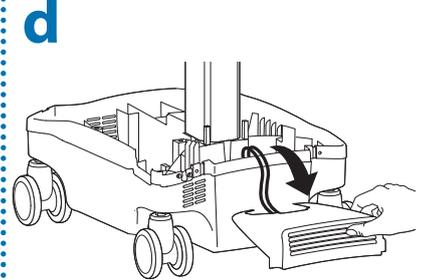

5 If your cart has circuit breakers, switch them both to "OFF".

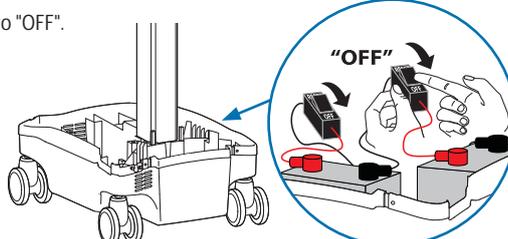

6

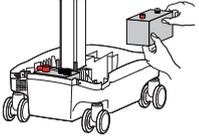




⚠ Caution: Replace only one battery at a time. Do NOT swap battery cables, doing so will cause sparking and trip the circuit breakers.




○ 10mm (3/8")
or
○ 8mm (5/16")

Depends on battery




a **⚠ Caution: Remove Black (-) before removing Red (+).**

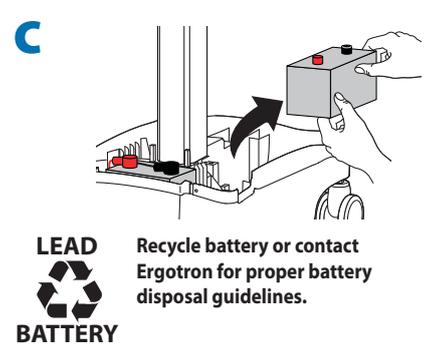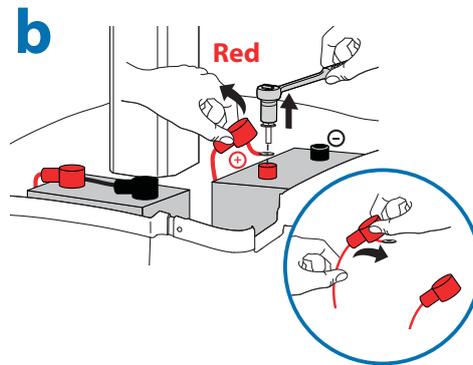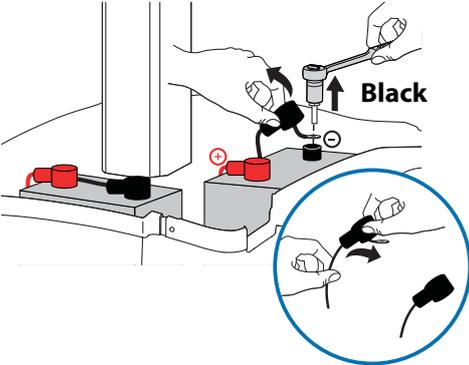

d



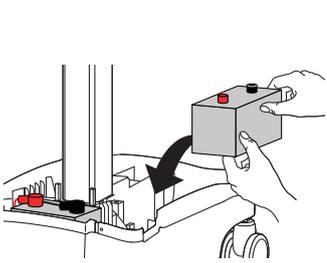


e **⚠ Caution: Connect Red (+) before connecting Black (-).**

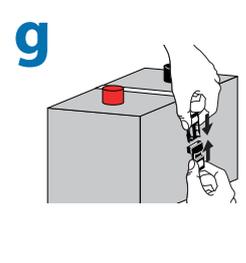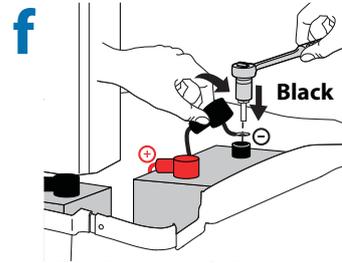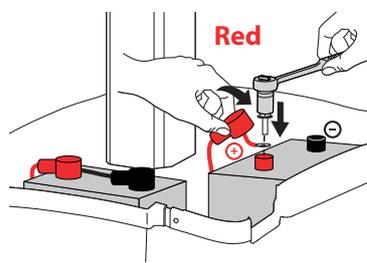


7




⚠ Caution: Remove Black (-) before removing Red (+).

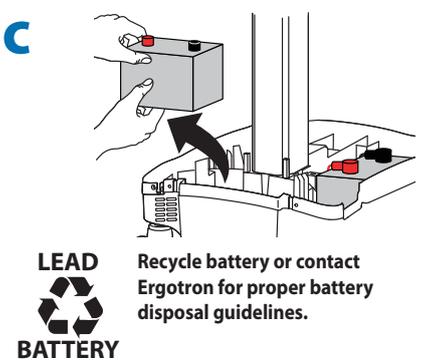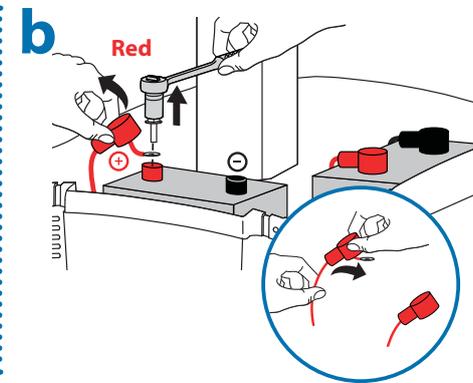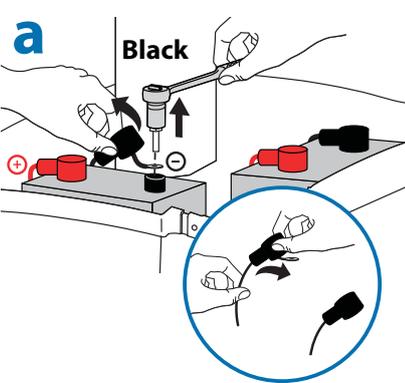

d

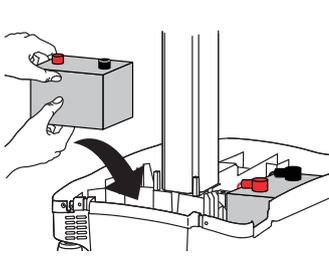



e **⚠ Caution: Connect Red (+) before connecting Black (-).**

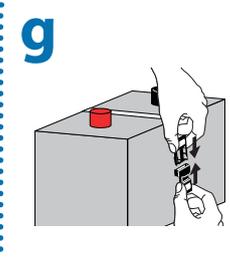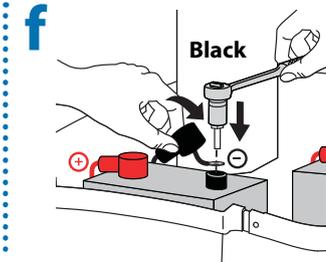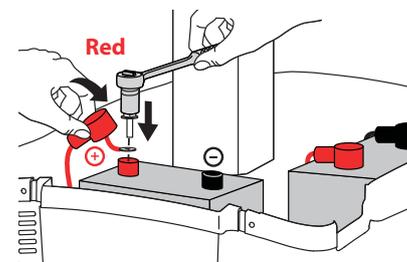


8

If your cart has circuit breakers, switch them both to "RESET".


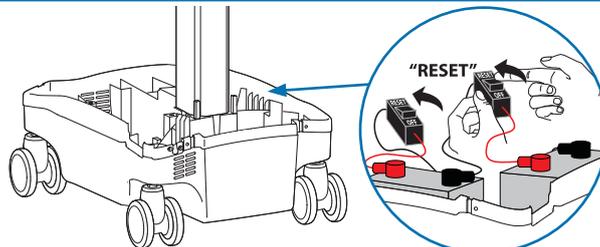


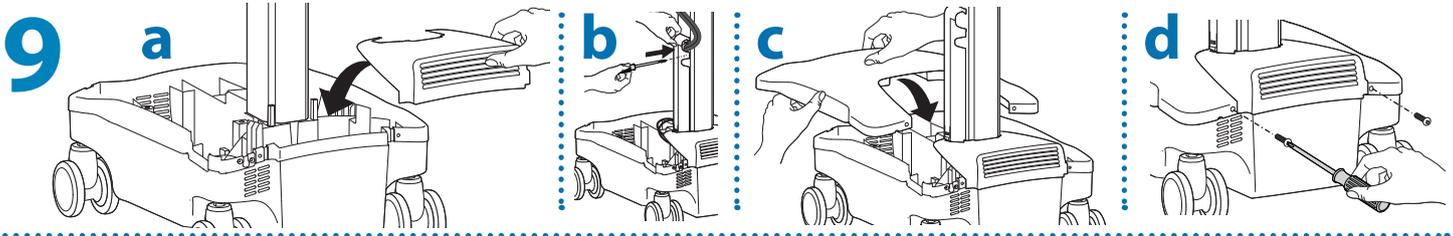

10 Initial Power on/Charge Battery (takes approximately 8 hours to charge)

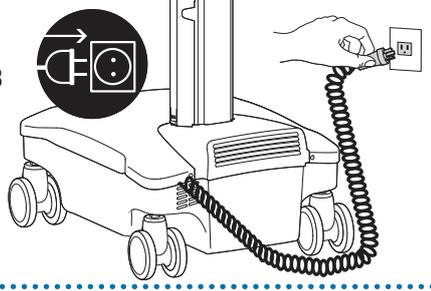

Plug Cart's Power Cord into wall outlet.



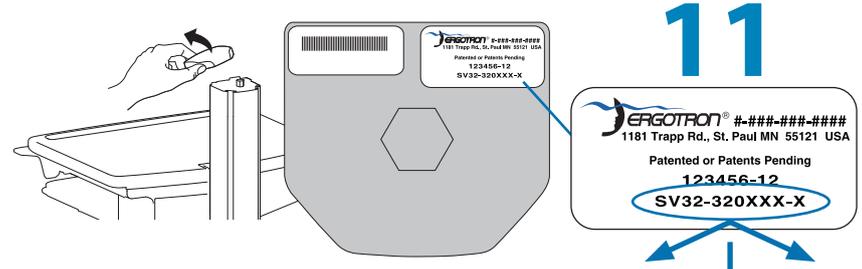
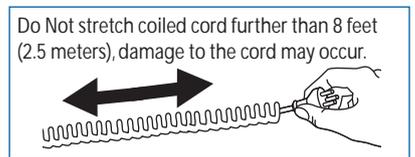


For carts with these part numbers only:

- SV32-32001-S
- SV32-32002-S
- SV32-32025-S
- SV32-32026-S
- SV32-32051-S
- SV32-32052-S

All other powered StyleView (SV32) carts

Turn on power system by holding power button down for 1 - 3 seconds.

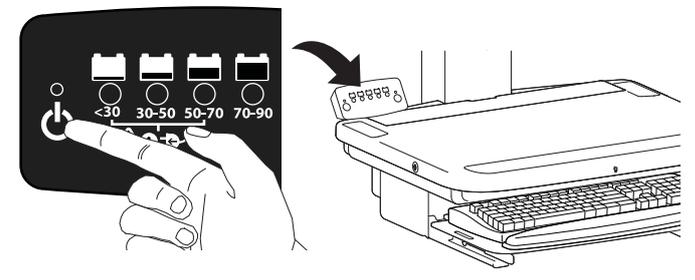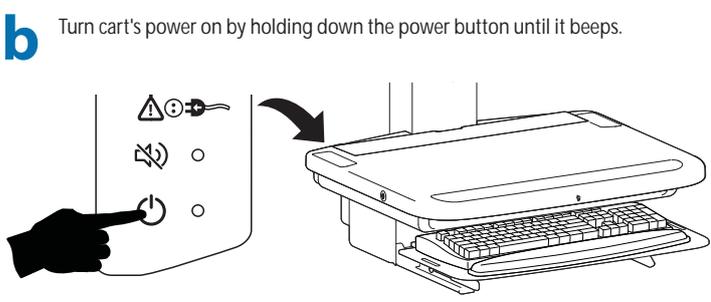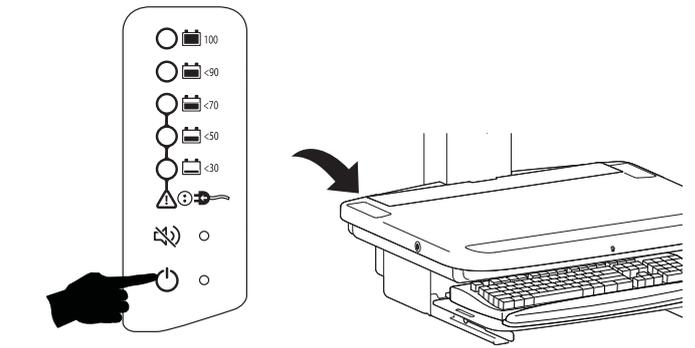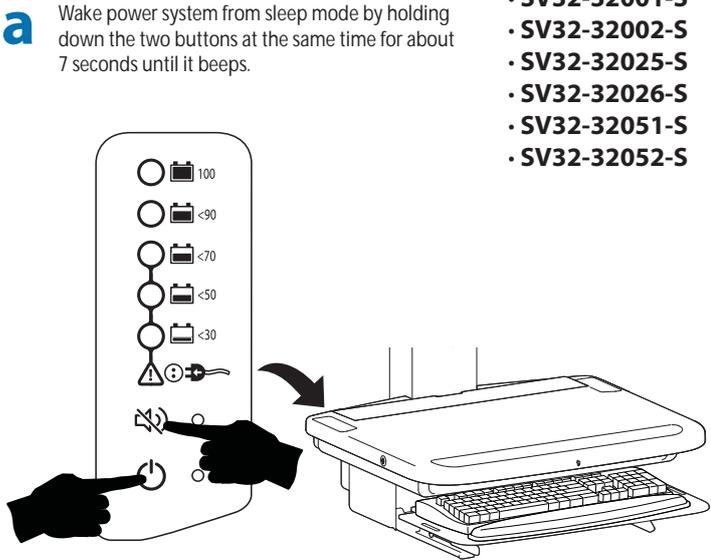

12 With cart's power cord plugged into the wall outlet, wait until cart is at 100% charge. (takes approximately up to 8 hours to charge)



Maintenance & Safety

Hazard Symbols Review

The Meaning of Symbols appearing in this Guide, on the Cart or on the Power System

These symbols alert you to a safety condition that demands your attention. You should be able to recognize and understand the significance of the following Safety Hazards if you encounter them on the Cart or within Cart documentation such as this User Guide.

Color	Symbol	Signal Word	Level of Hazard
Red		DANGER	Indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
Orange		WARNING	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
Yellow		CAUTION	Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury.
None		CAUTION	Used without the safety alert symbol indicates a potentially hazardous situation which, if not avoided, may result in property damage.
Red, Orange or Yellow		Electrical	Indicates an impending electrical hazard which, if not avoided, may result in personal injury, fire and/or death.

Ergotron Limited Warranty

Ergotron warrants that its products will remain free of defects in material and workmanship for five (5) years. All subject to the exceptions, exclusions and limitations as set forth below.

Exceptions to this warranty include:

- Three (3)-year warranty on all Neo-Flex™ by Ergotron products
- StyleView Carts:
 - Two (2)-year warranty on all power supply components by Ergotron
 - Battery carries an up to a one (1)-year performance warranty (see Limitations below) and six (6)-month warranty against manufacturer's defects. Note: In EU, the warranty against manufacturer's defects is two (2)-years.
 - One (1)-year warranty on all electronic non-power supply components (excludes AA batteries)
- OEM products resold by Ergotron are warranted by the respective manufacturer
- Product that is refurbished by Ergotron and sold by Ergotron carries a 90-day warranty
- Thirty (30)-day warranty on all service programs
- Ergotron warranties are nontransferable

Exclusive Remedy: If any Ergotron equipment is or becomes defective in material or workmanship during the warranty period, Ergotron will determine with the customer if the product defect is covered under warranty. Ergotron, at its sole discretion, may replace or repair the unit determined to be under warranty at a designated Ergotron location or at the customer location. The purchaser must get pre-approved authorization to return product to Ergotron. The Purchaser must contact Ergotron to obtain an RMA (Return Material Authorization) number. Ergotron will promptly repair or replace the item and ship it to Purchaser at no charge to the Purchaser. Performance of repair or replacement does not renew or extend the warranty period.

THIS REMEDY IS THE EXCLUSIVE REMEDY FOR BREACH OF WARRANTY AS TO THE EQUIPMENT, except in respect to personal injury. Purchaser shall give Ergotron immediate written notice of any personal injury resulting from use of Ergotron equipment.

Exclusions and Limitations: Ergotron disclaims liability for injuries or losses caused by or associated with installation or use of its equipment in any manner other than in strict conformance with the instructions set forth in its installation manuals, supplemental assembly and installation instruction sheets, technical bulletins or product literature, or by equipment that has been altered, damaged or misused.

Warranty on fully charged batteries left in an unused state for more than three (3) consecutive months is automatically void. Warranty on fully discharged batteries left in an unused state for more than three (3) consecutive days is automatically void.

Two (2)-year warranty on all power supply components by Ergotron; battery carries a 90 day performance warranty and six (6) month warranty against manufacturer's defects. Performance warranty is extended to 1 year on the battery providing:

- StyleLink software is installed and it shows compliance with exclusions and limitations
- Battery is not discharged below 25% capacity more than 25 times
- Battery is fully recharged at least once every 36 hours

Purchaser has not relied upon and Ergotron disclaims any and all warranties other than the expressed warranty stated above, subject to the exclusions and limitations provided herein. Ergotron disclaims all other warranties, expressed or implied, including but not limited to the warranties of merchantability and fitness for a particular purpose. Ergotron further disclaims liability to purchaser and all others for all consequential, incidental and special damages caused by or in connection with the products, their use or misuse, and their performance or nonperformance.

Americas Sales and Corporate Headquarters

St. Paul, MN USA
(800) 888-8458
+1-651-681-7600
www.ergotron.com
sales@ergotron.com

EMEA Sales

Amersfoort, The Netherlands
+31 33 45 45 600
www.ergotron.com
info.eu@ergotron.com

APAC Sales

Tokyo, Japan
www.ergotron.com
info.apac@ergotron.com

Worldwide OEM Sales

www.ergotron.com
info.oem@ergotron.com



© 2009 Ergotron, Inc.
All rights reserved.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.